Use the use for is not supplied individually. Only one copy of the instructions for use are provided per box and should therefore be kept in an accessible place where they are available upon request.

**NOTE:** Distribute instructions to all product locations and locations. These instructions contain important information for the safe use of the product. Read the instructions carefully and observe all precautions and warnings. Use the cautions before using this product. Failure to properly follow warnings, cautions and instructions could result in serious injury or death to the patient. Serious adverse events have been reported as a result of incorrect use.

**WARNING: A WARNING statement provides important information about a hazardous situation, which, if not avoided could result in death or serious injury.**

**CAUTION: A CAUTION statement provides important information about a hazardous situation, which, if not avoided could result in minor or moderate injury.**

**DESCRIPTION AND DESCRIPTION**  
PEP T-piece, Inhalation port -22M/15F-22M with one-way valve and mouthpiece, 22F, single patient use

Control the direction of flow of respiratory gases in breathing systems and anaesthetic circuits as an individual component.

This device may be used by adult and paediatric patients in hospital under the supervision of an appropriately qualified clinical personnel.

An Intersurgical PEP T-piece and mouthpiece is intended for use with Intersurgical PEP T-piece and mouthpiece. It is possible to add a PEP manometer connector (Intersurgical product code 1956020) between the exhalation port of this PEP T-piece.

This device may be used for up to 7 days, but not exceeding 50 cycles. Cleaning instructions are detailed in this instruction for use.

**PATIENT CATEGORY**  
This device may be used by adult and paediatric patients.

The product is available in one size only.

**PRE-USE CHECKS**  
Ensure the PEP T-piece and mouthpiece are undamaged and free of obstructions and foreign bodies including fibres.

1. Attach the mouthpiece to the PEP T-piece using a push and twist action (fig. 1).

2. Alternatively if a facemask is required instead of a mouthpiece, attach the PEP T-piece using a push and twist action (fig. 2).

3. Follow the instructions for use provided with the PEP resistor for assembly and directions for use. The PEP T-piece is configured with the PEP resistor 5.0 Ohm. When used with the positive expiratory pressure, the product is configured as detailed in figure 4.

4. Ensure that the patient is able to inhale and exhale through the device throughout use.

**CLEANING INSTRUCTIONS**  
Following each use, remove the PEP resistor and mouthpiece, or wash the PEP T-piece with a pull twist action (fig. 2).

5. Mix detergent in water to create washing solution and use to wash the T-piece and mouthpiece, then rinse in clean water. Wipe away excess water with a clean lint free cloth.

**NOTE:** The maximum number of uses and wash cycles is 50, over a maximum of 7 days.

6. The device shows signs of degradation or is damaged and no longer suitable for intended use.

7. This product is to be used in hospital under the instruction of appropriately qualified medical personnel.

8. Following each use, remove the PEP resistor and mouthpiece, or wash the PEP T-piece with a pull twist action (fig. 2).

9. For USA, Rx Only: Federal law restricts this device to sale by or on the order of a physician.

**WARNINGS**  
Use of the medical device requires full understanding and observation of the instructions for use.

10. The medical device may only be used for the purpose specified as the intended use. The clinician should be aware of and consider any known pre-existing conditions that may affect the safe use of the product.

11. Observe all caution and warning statements and all other instructions for use and observe cautions and warnings for all attached products.

12. Observe all caution and warning statements in these instructions and all other instructions for use and observe cautions and warnings for all attached products.

13. Following cleaning ensure the product has dried completely before fibres are attached to the device.

14. Do not use product past the expiry date.

**STORAGE**  
Store in a cool, dry place at room temperature over the indicated product shelf-life.

**DISPOSAL**  
This device and the product must be disposed of in accordance with local hospital infection control and waste disposal regulations.

Following healthcare use, dispose in accordance with local regulations.

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